

Case Study - Burn

Week 0



Partial Thickness Burn onset: Jan. 7, 2006

Study Begin: Feb 21, 2006

Wound Dimensions: 57.9 cm², no depth (VisiTrak[®])

Dressing applied: SelectSilver[™], gauze, Kerlix[®] and Coban[™]

Much pain, foul odor, moderate drainage.

Week 1



Date: Feb 28, 2006

Wound Dimensions: 44.7 cm², no depth (VisiTrak[®])

Dressing applied: Mepitel[®], SelectSilver[™], gauze, Kerlix[®] and Coban[™]

Less pain, no odor, moderate drainage. Mepitel[®] used as contact layer to prevent sticking.

Week 2




Date: March 7, 2006

Wound Dimensions: 38.7 cm², no depth (VisiTrak[®])

Dressing applied: Mepitel[®], SelectSilver[™], gauze, Kerlix[®] and Coban[™]

Some pain, no odor, mild drainage. Mepitel[®] used as contact layer to prevent sticking.

Week 4

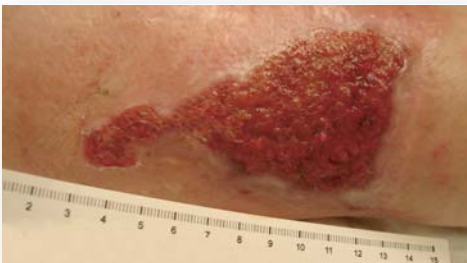

Date: March 31, 2006

Wound Dimensions: 25.1 cm², no depth (VisiTrak[®])

Dressing applied: Patient off study. Physician continuing with Mepitel[®], SelectSilver[™], gauze, Kerlix[®] and Coban[™].

Pain scale 1 (1-10), moderate drainage. Mepitel[®] used as contact layer to prevent sticking.